
- 1 Kontrolllampe / control lamp
- 2 Betjeningsknapp / control button
- 3 Ladelampe / charging lamp
- 4 Ledning, plugg / cable, connector
- 5 Kabelinngang / input socket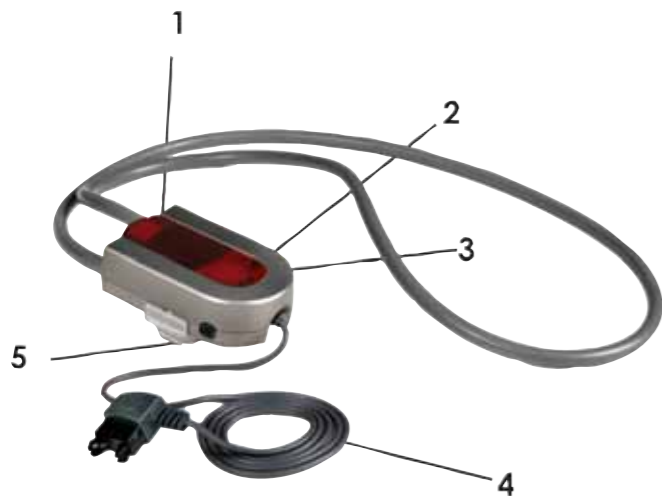

N - Huskeliste

På: Trykk inn betjeningsknappen (2) i ca. 1 sek.

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Answer and terminate a call by pressing the button briefly.

Switch on or off Dialogue Mode by pressing the button for approximately two seconds.

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Commidt specialises in handsfree telephone solutions for
people hard of hearing. Our mission is to provide innovative
multiple-use products to improve communication and quality
of life for people hard of hearing.

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Amplified neck loop with integrated voice amplifier
for telephones and audio equipment.

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Bluetooth amplified neck loop with integrated voice
amplifier for telephones and audio equipment.

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Slim T-Hook with answering push button
for telephones.

Halsslynge for telefoner og audioutstyr.
Multi turn neck loop for telephones and
audio equipment.

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INDEX

Getting started	12
Use as neck loop for telephone	13
Use as Voice Amplifier	14
Changing the battery	15
ACCESSORIES	16
Problem solving	17
Check list	18

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CLASSIC

Amplified neckloop with integrated voice amplifier for telephones and audio equipment.

Please read this user manual carefully before using your Classic

Please refer to the user manual of the equipment you would like to connect to your Classic before use.

Preparation:

- Insert the charging plug in the unit (5) and connect to a main socket.
- Normal charging time is two-three hours. This is controlled automatically to ensure optimal battery charging.
- The charging lamp (3) lights continuously during charging.
- The charging lamp (3) turns off when the battery is fully charged.
- Low battery voltage is indicated by flashing charging lamp (3) as well as by an audible signal in the Neckloop.

Getting started:

Switch ON the unit by pressing the control button (2) for at least one second: an audible signal will be heard and the control lamp (3) will light up. A faint flash in the control lamp (1) every fourth second indicates that the unit is ON.

To switch OFF, press control button (2) for at least four seconds. You will then hear an audible signal and the control lamp (1) will flash.

Used as neckloop with telephone:

Connect the unit and the mobile by means of the cable/connector (4).

Switch the hearing aid(s) to T or MT.

Switch on Classic by pressing control button (2) for at least one second.

- Making a call

Make a call using the telephone in the normal manner. The sound will be heard through the neckloop.

To terminate the call, press the control button (2) or hang up the telephone in the normal manner. The control lamp (1) will give a short flash and an audible signal will be heard (phone dependent).

An incoming call will be indicated by a ringing tone in the hearing aid. The control lamp will flash repeatedly while the phone is ringing (phone dependent). Answer/terminate the call by pressing control button (2).

- Rejecting incoming calls

When the telephone rings and you do not wish to answer the call, hold button (2) for at least two seconds (gives the busy signal).

- Voice dialling

See the user's manual of your mobile phone for instructions on recording names for voice dialling.

Press button (2) briefly; an audible signal will indicate that the unit is ready for voice dialling. Say the name of the person you wish to call and the telephone will dial the number automatically.

- Adjusting the sound level

The sound level is phone dependent. Please see the user's manual of your telephone.

Use as a Voice Amplifier (Dialogue Mode)

To set the neckloop to Dialogue Mode (voice amplifier), press the button (2) for approximately two seconds. You will hear a short acknowledgement signal and the control lamp (1) will also flash. Deactivate the Dialogue Mode function by pressing control button (4) briefly.

While the unit is in Dialogue Mode, you can answer or reject incoming calls in the normal manner.

When the telephone rings while the neckloop is in Dialogue Mode, Classic will automatically switch to Telephone Mode (phone dependent)

When in normal use, the internal microphone will be in use, but you can also connect an external microphone to increase the sound. The internal microphone will then be switched off.

Use as an audio equipment amplifier; Please see accessories for more information.

This manual may differ because of the telephone model

Shirt/blouse Clip

The enclosed clip is pressed in place at the back of the unit and can be attached in both directions. COMMidt recommends using the clip in order to decrease noise when you are moving.


Connector guide

Please visit our website www.commidt.com to find compatible mobile phones for your Neckloop. The Connector guide will be updated at all times.

NB! Choose the type of mobile phone (make) at the bottom of the page.

Changing the battery

- Unscrew the two screws on the back of the unit
- Remove the front panel
- Note how the battery and battery cable are positioned
- Pull out the battery plug (6)
- Remove the old battery
- Insert a new battery and position the battery cable as the old one was, before inserting the plug
- Replace the front panel and secure with the screws.

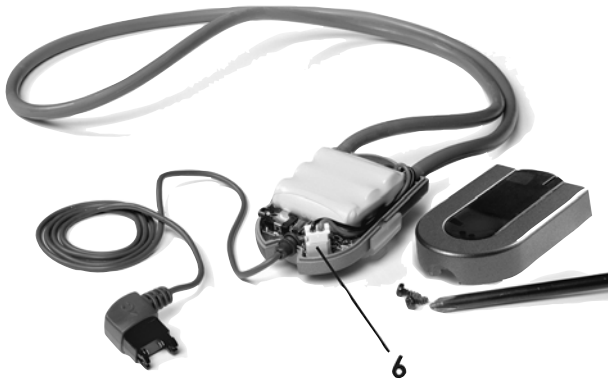Battery type: Ni-MH, 3.6 V 300 mAh

Old batteries should be desposed of at an approved reception point for recycling.

AVOID unnecessary contact with the unit's circuit board.

Accessories

A selection of our accessories – Please visit our web site for information www.commidt.com

External Microphone

Use of an external microphone will improve the quality of the sound you transmit, and will also enable you to wear the unit under your clothing. Connect the external microphone to the unit (5).

Note that when the external microphone is in use, the internal microphone is switched off!

It is important to bear this in mind, especially when the telephone rings.

Microphones:

- **Clip-on microphone** (Article No. 901) with 40 cm lead for collar attachment. The use of an external microphone reduces unwanted background noise. It can also be used when wearing the unit under clothing.
- **Clip-on microphone** (Article No. 902) with 1.5 m lead for attachment to a conversation partner's collar (for example in a car) when using Dialogue Mode.
- **Directional microphone** (Article No. 903) with 60 cm lead for use in Dialogue Mode. Also available with 3 metre lead and table stand (for example in meetings). Has two settings: Spot (marked MO) and Wide (marked ST). In the Spot setting, the microphone should be pointed towards the source, for optimal sound quality. In the Wide setting, all sound originating close to the microphone will be amplified.

Car Charger (Article No. 907):

The car charger will charge the unit in the same way as the normal charger.

Audio Cable (Article No. 908):

This can be used to connect an MP3 player, CD player, PC, two-way radio, cordless telephones or similar unit to the input socket (5). You can then listen to music using the Dialogue function of the neckloop. The cable is 65 cm long and can be extended using standard extension cables if necessary.

IMPORTANT: Only this cable, which is specially made for this purpose, **can be used to connect such equipment.** When the telephone rings, this cable must be unplugged from the unit. This is to avoid the sound from the audio unit to be transferred to the person making the call.

Battery (Article No. 951):

Battery pack Ni-MH 3.6/300, 3.6 V 300 mAh

Maintenance:

Keep the unit clean by wiping with a dry cloth.

Do not expose the unit to moisture.

Technical Tips:

Problem solving:

1. The unit cannot be switched on.
Charge the battery. If this doesn't help, replace the battery pack and re-charge. A short charge life indicates that the battery is old and should be replaced.
2. The battery won't charge.
Check that the charging lamp (3) lights when the plug is inserted. Check that the plug (5) is correctly inserted into the main socket and that the main socket is live.
3. The control button doesn't work. Reset the unit by pressing the control button (2) for approximately eight seconds.
4. The light indicator doesn't work. Reset the unit by pressing the control button (2) for approximately eight seconds

Technical Data

Battery type: Ni-MH, 3.6 V 300 mAh

Charging time: approx. 2-3 hours

Call time: approx. 48 hours

Standby time: approx. 200 hours

Check List

Switch on: Press the control button (2) for approximately one second.

Switch off: Press the control button (2) for approximately three seconds.

Answering calls: Press the control button (2) for approximately one second.

Rejecting calls: Press the control button (2) for at least two seconds.

Terminating a call: Press control button (2).

Voice dialling: Press control button (2) briefly once to prepare the unit for voice dialling.

Dialogue Mode: Press control button (2) for two seconds.

Simplified Operation

Switch on or off the unit by pressing and holding the control button.

Answer and terminate a call by pressing the button briefly. Switch on or off

Dialogue Mode by pressing the button for approximately two seconds.

When the charging lamp flashes, recharge the battery.

Electrical and electronic equipment (EEE) contains materials, components and substances which can be dangerous and detrimental to human health and the environment if waste electrical and electronic equipment (WEEE) is not disposed of correctly.

Products marked with the “crossed-out waste bin” symbol shown below are electrical and electronic equipment. This symbol indicates that waste electrical and electronic equipment must not be discarded along with unsorted household waste, but must be collected separately.

In the development and manufacture of your COMMidt product, high-quality materials and components have been used which can be recycled. When disposing of electrical and electronic equipment you should therefore make use of a collection arrangement intended for this purpose, which may involve either delivery at your local collection point or recycling station or collection at your home. Further information should be obtained from your local authorities.