Sounds better

COMMidt · JAZZ

Safety and maintenance information

INTERFERENCE:

-All electronic equipment may from time to time be susceptible to interference, which could affect the performance. Local interference of e.g. PC, TV, luminous tube etc may cause noise. To avoid this interference, try to move away from the noise source.

USAGE

Read the user's manual carefully before using this product. Performance may differ. Use only in the normal position as explained in the user's manual.

QUALIFIED SERVICE

Only qualified personnel may repair this product. Only open the unit when changing the battery.

WATER-RESISTANCE

This unit is not water-resistance. Keep it dry and clean by wiping with a dry cloth.

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Betjeningsknapp / control button

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JAZZ

T-hook for telephones or audio equipment.

Getting started:

Connect Jazz and the telephone or the audio equipment by means of the cable/connector. Switch the hearing aid(s) to T or MT.
The functionality of Jazz is dependent on the connected telephone or audio equipment.

Use as T-hook with telephone:

• Making a call

Make a call using the telephone in the normal manner. The sound will be heard through the neckloop.

To terminate the call, press the button on the microphone enclosure or hang up the telephone in the normal manner. An incoming call will be indicated by a ringing tone in the hearing aid (phone dependent). Answer the call by pressing button. Terminate the call by pressing the button.

• Rejecting incoming calls

See the user's manual of your mobile phone.

• Voice dialling

See the user's manual of your mobile phone.

Refer to the user's manual of the telephone for further details.

Jazz connected to audio equipment:

Sound will be transmitted from the audio equipment. Use and functionality are dependent on the connected equipment.
Refer to the user's manual of the audio equipment for further details.

Maintenance:

Keep the unit clean by wiping with a dry cloth.
Do not expose the unit to moisture.

Connector Guide

Please visit our website www.commidt.com to find compatible mobile phones for your Neckloop. The Connector guide will be update at all times.
NB! Choose the type of mobile phone (make) at the bottom of the page.